

Portable Resuscitator



L175-140R (cylinder not included)

The LSP Portable Resuscitator is a fast, simple, and effective system to resuscitate a non-breathing patient while performing CPR, or to provide 100% oxygen to a breathing patient on demand with minimal inspiratory effort. There are no adjustments to make or dials to set. Simply connect the desired resuscitation device and open the cylinder valve, and the resuscitator is ready for use.

The demand valve is designed to resuscitate a non-breathing patient, or to provide assisted breathing with 100% oxygen on a conscious patient. To use for resuscitation, the caregiver simply presses the button on the demand valve, to inflate the lungs. When the button is released, the gases are expelled from the lungs through the exhalation (non-rebreathing) valve. For the demand function, the patient simply needs to hold the mask to his mouth and breathe. When the patient initiates a breath, the demand valve will sense the negative pressure and assist the breath with 100% oxygen.

The low flow selections on the included oxygen regulator provide accurate low flow delivery from 0-25 lpm. The reusable venturi aspirator uses the oxygen flow to provide up to -400 mm Hg at 30 lpm for effective emergency aspiration of the oral/nasal area. The unit comes with a replaceable 8 ounce polypropylene aspirator jar.

The kit is housed in a highly-visible orange plastic case with an integrated handle. The case is designed to securely hold either a D (M-15, 425 liters) or a Jumbo D (M-22, 640 liters) oxygen cylinder. This lightweight case is buoyant and will float in water for a short period of time, allowing for immediate retrieval. The two-piece hinged door allows easy access to the oxygen regulator and demand valve, as well as the cylinder valve. Made of high-density polyethylene plastic with recessed stainless steel hinges, the case is impervious to corrosive materials, chemicals, and adverse weather conditions. Hinges, latches and the plastic case are designed to withstand rugged use and provide years of trouble-free service. They are guaranteed to be free of workmanship defects for a period of two years. When necessary, latches are easily replaced. An optional rugged yet lightweight Soft Pack Case is available (L040124).

The LSP Portable Resuscitator is a quickly accessible and efficiently organized airway management system designed for the EMS professional.

Portable Resuscitator

SPECIFICATIONS

Regulator Supply Pressure: 40-60 psig at 2200 psig

For full specifications on flow rates, see the sell sheets for the LSP EMT Resuscitator and LSP Demand/Resuscitator Valve.

Regulator Inlet Fitting: CGA 870 (Pin-Index Yoke)

Case Dimensions: Plastic Case: 8" x 9.5" x 23.5" (20.3 x 24.1 x 59.7 cm);

Soft Case: 10" x 9" x 24" (25.4 x 22.9 x 60.9 cm)

Complete Kit Weight: approximately 17-19 lbs. (7.7-8.6 kg), depending on equipment included.

ORDERING INFORMATION

L175-140R includes

EMT Resuscitator L576
Oxygen Regulator L270-220
Reusable Aspirator L146-020
Child Airway
Adult Airway
Adult Oxygen Mask w/7' tubing 64041
Adult Cuffed Mask L099-005
2 Oxygen Hoses, 6', green L535026
Handwheel with chain L002865-050
Molded Plastic Case L040088

L175-140R-CE includes

Oxygen Regulator L270-220-CE
Adult Oxygen Mask w/7' tubing 64041
Oxygen Hose, 6', white
Remaining items same as L175-140R

Optional Case

Soft Pack Case L040124

ORDERING INFORMATION

L175-01R includes

Demand/Resuscitator Valve L063-05R
Oxygen Regulator L270-220
Reusable Aspirator L146-020
Child Airway
Adult Airway
Adult Oxygen Mask w/7' tubing 64041
Adult Cuffed Mask L099-005
2 Oxygen Hoses, 6', green L535026
Handwheel with chain L002865-050
Molded Plastic Case L040088

L175-010 includes

Demand/Resuscitator Valve L063-050
Remaining items same as L175-01R

L175-010D includes

Demand/Resuscitator Valve L063-050
Oxygen Regulator 32-29-2550-CE
Aluminum D Cylinder (empty)
Remaining items same as L175-01R

ORDERING INFORMATION

L111-010 includes

Demand/Resuscitator Valve L063-050
Oxygen Regulator L735-260
Reusable Aspirator L146-020
Child Airway
Adult Airway
Adult Cuffed Mask L099-005
2 Oxygen Hoses, 6', green L535026
Handwheel with chain L002865-050
Molded Plastic Case L040088



L576
L270-220-CE
L099-005
L535026
L063-05R
L063-050
32-29-2550-CE
L735-260



L576 EMT Resuscitator shown with L099-005 Adult Cuffed Mask and L535026 Oxygen Hose



L270-220 Oxygen Regulator



L040088 Molded Plastic Case

L146-020 Reusable Aspirator shown with L535026 Oxygen Hose



L040124 Soft Pack Case



EMERGO EUROPE
Prinsessegracht 20
2514 AP The Hague
The Netherlands



Allied Healthcare Products, Inc. ▪ 1720 Sublette Avenue ▪ St. Louis, MO 63110 USA
314/771-2400 ▪ 800/444-3954 ▪ FAX: 314/771-0650 ▪ www.alliedhpi.com

U.S.
Toll Free: 800/444-3940
FAX: 800/477-7701

Canada & Mexico
Toll Free: 800/446-0552
FAX: 800/246-6201

International
Telephone: 314/268-1683
FAX: 314/771-5183

ISO 13485: 2003

©Allied Healthcare Products, Inc.®

All specifications are nominal and subject to change without notice.

Form 90-00-0025
Rev. May 2017