

# *EMT Resuscitator*



L576-020

The LSP EMT Resuscitator is today's most advanced design in oxygen powered manually triggered breathing devices. Designed to deliver 100% oxygen to a breathing or non-breathing patient, the EMT Resuscitator provides emergency care professionals with greater versatility, increased safety and improved performance over existing hand-held devices.

Designed to operate with a regulated oxygen supply of  $50 \pm 5$  psig ( $344 \pm 34$  kPa), the EMT Resuscitator immediately terminates both pressure and flow at  $60 \text{ cm H}_2\text{O}$ . As a secondary pressure safeguard, if the primary pressure limit is exceeded for any reason, the secondary spring-loaded relief valve is designed to prevent pressure from exceeding  $80 \text{ cm H}_2\text{O}$ . The EMT Resuscitator may be attached to a portable pressure-regulated oxygen cylinder or a central oxygen source. It may be used with a mask, endotracheal tube or tracheotomy tube.

Advanced pneumatic technology limits and regulates gas flow to 40 lpm during the manually-triggered breath, but will allow flow rates of up to 115 lpm during a spontaneous breath. Addressing AHA guidelines, the EMT Resuscitator delivers a constant pre-set flow of 40 lpm, reducing the possibility of gastric insufflation during mask ventilation and improving the distribution of gas throughout the lung when ventilating through an endotracheal or tracheotomy tube.

The EMT Resuscitator features built-in anti-asphyxiation ports that allow the patient to breathe outside ambient air in the event of source gas failure. In addition, a  $360^\circ$  swivel oxygen inlet attachment eliminates the problem of line twisting and tangling.

## **Key Features**

- Easy to clean and service ... disassembles into three basic components for cold chemical disinfecting and reassembly
- Versatile ... provides administration of low flow oxygen and/or oxygen powered resuscitation.
- Durable, compact and lightweight for years of continual cost-effective performance



# EMT Resuscitator

## SPECIFICATIONS

**Supply Pressure:** 50±5 psig (344±34 kPa)  
**Pressure to Initiate a Breath:** 0 to -2.5 cm H<sub>2</sub>O  
**Manually Triggered Flow:** 40±5 lpm @ 50 psig (344 kPa) supply  
**Demand Flow:** 100 lpm @ 0 to -5 cm H<sub>2</sub>O pressure  
**Peak Demand Flow:** 115±10 lpm @ 50 psig (344 kPa) inlet; -5 cm H<sub>2</sub>O or less to obtain 100 lpm  
**Pressure Limit:** 60±5 cm H<sub>2</sub>O  
**Secondary Pressure Relief:** 65-80 cm H<sub>2</sub>O maximum  
**Expiratory Resistance:** less than 2.5 in. (6.35 cm) H<sub>2</sub>O @ 160 lpm flow  
**Anti-Asphyxiation Entrainment:** 55 lpm minimum @ -5 cm H<sub>2</sub>O pressure  
**Storage Temperature:** -40° F to 160° F (-40 C to 71.1 C)  
**Operating Temperature:** -30° F to 125° F (-34.4 C to 51.7 C)  
**Inlet Fitting:** standard male oxygen DISS (swivel)  
**Filter:** 25 micron wire mesh  
**Outlet:** 22 mm outside diameter x 15 mm inside diameter  
**Materials:** Body, cap & button - liquid crystal polymer; outlet - polysulfone; inlet fitting - plated brass

## ORDERING INFORMATION

L576	EMT Resuscitator
L576-010	EMT Resuscitator with 6' (1.8 m) oxygen hose
L576-020	EMT Resuscitator with 6' (1.8 m) oxygen hose, adult cuffed mask
L099-004	Adult Cuffed Mask, reusable, size 4
L099-005	Adult Cuffed Mask, large, reusable, size 5
L535026	Oxygen Hose, 6' (1.9 m), with DISS fittings

For information on international configurations,  
contact an Allied Healthcare international sales or customer service representative.



L576-020 EMT Resuscitator with L535026 oxygen hose and  
L099-004 adult cuffed mask



**EMERGO EUROPE**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands



Allied Healthcare Products, Inc. ▪ 1720 Sublette Avenue ▪ St. Louis, MO 63110 USA  
314/771-2400 ▪ 800/444-3954 ▪ FAX: 314/771-0650 ▪ www.alliedhpi.com

U.S.  
Toll Free: 800/444-3940  
FAX: 800/477-7701

Canada & Mexico  
Toll Free: 800/446-0552  
FAX: 800/246-6201

International  
Telephone: 314/268-1683  
FAX: 314/771-5183

**ISO 13485: 2003**

©Allied Healthcare Products, Inc.®

All specifications are nominal and  
subject to change without notice.

Form 90-00-0010  
Rev. May 2017