


# ResusciTIMER®

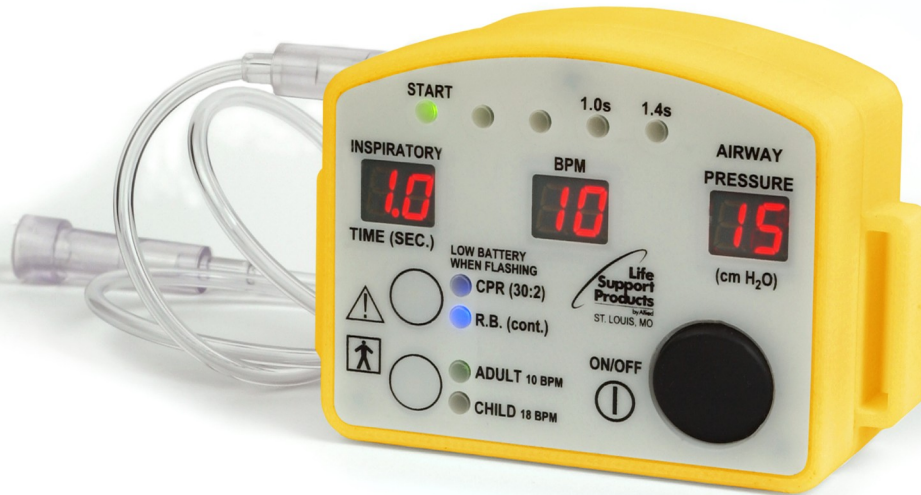

## *Perfect Ventilation Technique*

With the innovative ResusciTIMER, experienced EMS professionals can confirm proper manual ventilation technique. EMTs in training can improve their skills. In fact, anyone can learn with just minutes of basic instruction. This easy- to-use device provides real-time monitoring and feedback of breaths per minute, inspiratory time and airway pressure to guide in resuscitation timing and technique.


With two modes of operation ... CPR and Rescue Breathing, and two patient modes ... adult and child, it monitors the pressure and flow from the BVM and provides visual and audible cues to guide CPR or rescue breathing to meet AHA guidelines. It can be attached to virtually any disposable or reusable bag valve mask.

The ResusciTIMER is an affordable and effective tool for mass casualty preparedness, everyday transport, or training. This simple yet effective device assures caregivers of every skill level that they are providing manual ventilation with perfect technique.



ResusciTIMER shown attached to an LSP L770 adult bag valve mask with the L770-CPR-VEL Velcro® Strap

# ResusciTIMER®

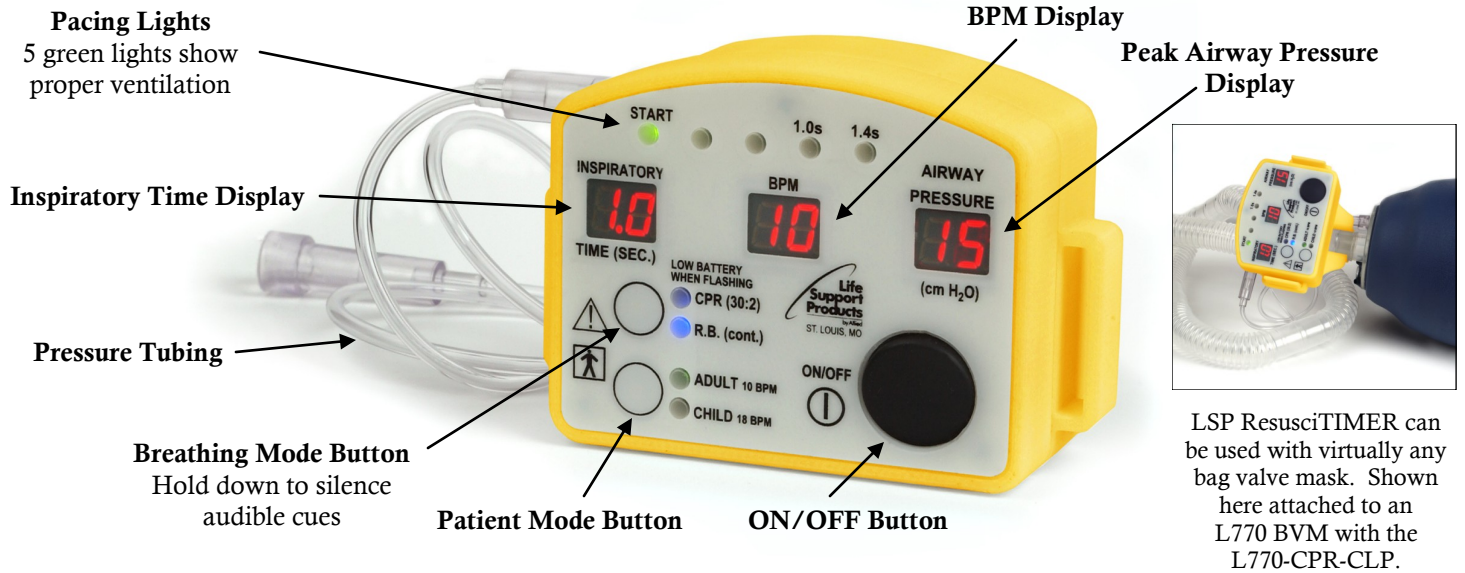

LSP ResusciTIMER can be used with virtually any bag valve mask. Shown here attached to an L770-BVM with the L770-CPR-CLP.



L770-CPR-ADP



L770-CPR-CLP



L770-CPR-VEL



L770-CPR-PWR

## SPECIFICATIONS

- Weight:** 5 oz. (including battery)
- Dimensions:** 2.7" H x 3.9 W x 1.4" D (6.8 cm H x 9.9 cm W x 3.5 cm D)
- Battery:** 9V alkaline battery
- Battery Life:** 4.5 hours continuous use @ 72° F
- Alternate Power Source:** 9V power supply with battery door
- Airway Pressure:** 60 cm H<sub>2</sub>O maximum; accuracy 5%
- Adult Setting:** 1 time = 1.0 - 1.4 seconds 10 ± 2 BPM
- Child Setting:** 1 time = 1.0 - 1.4 seconds 18 ± 2 BPM
- Operating Conditions:** Temperature: 0° F to 120° F (10° C to 38° C)  
Relative Humidity: 0% to 90%
- Storage Conditions:** Temperature: -20° F to 140° F (-29° C to 60° C)  
Relative Humidity: 0% to 90%
- Warranty:** One year limited warranty

## ORDERING INFORMATION

**L770-CPR** ResusciTIMER®, pressure tube assembly, in-line pressure tube adapter, Velcro® strap, 9V battery, instruction manual

## ACCESSORIES

- L770-CPR-ADP** Airway Pressure Adapter with filter and tube assembly; single use only (15/case)
- L770-CPR-TUBE** Pressure Tube Assembly; in-line pressure tube adapter for BVMs without manometer port; single use only (15/case)
- L770-CPR-CLP** BVM Hose Clamp, for attachment to LSP L770 BVMs
- L770-CPR-VEL** Velcro® Strap, for attachment to most BVMs other than LSP L770 BVMs (15/case)
- L770-CPR-PWR** 9 V Power Supply with battery door; for non-field use when training



Allied Healthcare Products, Inc. • 1720 Sublette Avenue • St. Louis, MO 63110 USA  
314/771-2400 • 800/444-3954 • FAX: 314/771-0650 • www.alliedhpi.com

U.S.  
Toll Free: 800/444-3940  
FAX: 800/477-7701

Canada & Mexico  
Toll Free: 800/446-0552  
FAX: 800/246-6201

International  
Telephone: 314/268-1683  
FAX: 314/771-5183

ISO 13485: 2003

©Allied Healthcare Products, Inc.®

All specifications are nominal and subject to change without notice.

Form 90-00-0009  
Rev. January 2013