


## *Floor-Mounted Pedestal*

### Elements

- Eliminates hoses and wires that clutter the floor
- Provides ready access to medical gas and electrical services near the point of use
- Provides remote receptacles for EEG/EKG, 115 volt electrical service, and ground jack receptacles in addition to the medical gases, including high pressure nitrogen
- Strong 18-gauge stainless steel shell
- UL listed

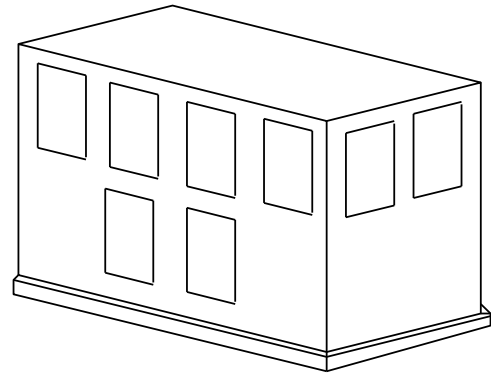

### Description

Where indicated on the drawings, provide stainless steel floor pedestal. Coordinate with other trades to avoid conflict with other services.

The pedestal shall be constructed of 18-gauge stainless steel with No. 4 finish. All welds are to be ground smooth for seamless appearance. The mechanical contractor shall provide medical gas piping. Access panel shall be furnished in each pedestal to facilitate installation and servicing of internal fixtures.

The floor pedestal can be designed to the installers required dimensions. The size of the pedestal will dictate the available locations for medical gas and electrical outlets. A stainless steel collar shall be provided to trim out the pedestal against the finished floor.

Final connection to the medical gas outlets shall be made by the installer.

The floor pedestals can be equipped with the following services:

- Medical gas outlets (side mounted)
  - Quick-Connect
  - DISS (Diameter Index Safety System)
- Electrical services (side mounted)
  - Hubbell 23000 HG, 115V AC receptacles
  - Hubbell 8200, 15/20 amp receptacles
  - 5-terminal receptacles, Amphenol
  - Hubbell 8300, GFCI 15/20 amp receptacles #MS-3100A-14S-5S, for EEG/EKG
  - Hampden SLR-3S ground jack receptacles
  - RJ data jack/telephone jack

Note: Internal wiring of electrical devices shall be the responsibility of the electrical contractor.

Note: Per NFPA revision 2012, any pedestal installed in a designated wet location must be equipped with one GFCI receptacle per electrical circuit.

## Ordering Information - Floor-Mounted Pedestal

Chemetron Floor-Mounted Pedestals are manufactured to customer specifications. The following steps must be completed prior to the manufacture of the floor-mounted pedestal.

### Step 1

General information to remember when selecting the devices in a pedestal:

- Use of EEG/EKG remote monitoring with electric power receptacles is not recommended in pedestals without properly shielded monitor cable. Consult monitor manufacturer.
- Specify color for electrical outlets.
- Monitor receptacles should be wired under monitor manufacturer's recommendations for proper shielding.
- Adjacent electrical receptacles will be provided in double, triple or quadruple configurations with common back box and cover. Adjacent ground receptacles or other adjacent electrical receptacles of the same type will be furnished with common back box and cover plate for installation convenience.
- Wiring to factory installed electrical receptacles shall be provided by others.

### Step 2

Indicate the size and quantity of the pedestal.

Quantity \_\_\_\_\_

Height \_\_\_\_\_ x Width \_\_\_\_\_ x Depth \_\_\_\_\_

### Step 3

Select the style of medical gas outlet installed in the pedestal. If nitrogen is used in the pedestal, indicate the style of outlet to be installed.

_____ Chemetron® Quick-Connect	_____ Oxequip® Quick-Connect	_____ <u>Nitrogen</u>
_____ Connect2® Quick-Connect	_____ DISS	_____ DISS
_____ Puritan Bennett style quick-connect		_____ Dyna-Con
		_____ Schrader


### Step 4

The drawing on the next page shows the available locations on the pedestal. Symbols on the drawing indicate where various accessories may be located. Select the accessories desired and the location on the column from the table shown on the next page.




**Installation Instructions**

**Pedestal with dimensions other than 12" wide x 12" deep**

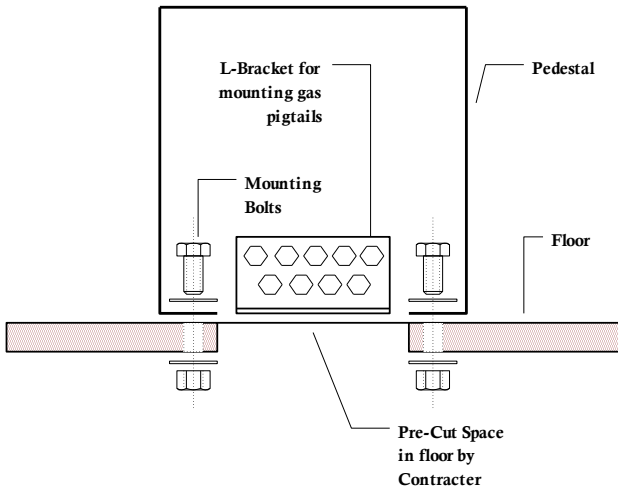

**Pedestal with dimensions 12" wide x 12" deep**

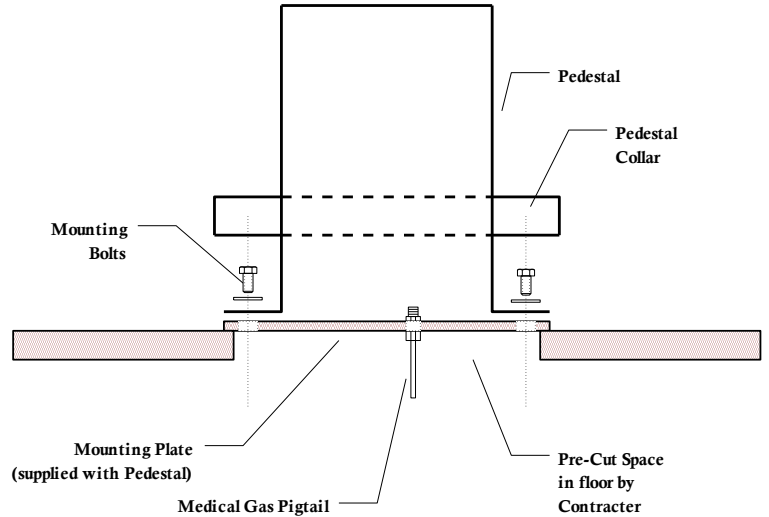

**NOTE:**

- Pedestal collars and mounting plates are provided only with units that have dimensions of 12" wide x 12" deep.
- Gas pigtailed are provided for each medical gas outlet for all size pedestals.
- L-bracket is provided to mount gas pigtailed in pedestals that have dimensions other than 12" wide x 12" deep.
- The electrical devices are not wired.

All specifications are nominal and subject to change without notice.

Warranty:  
See Allied Statement of Warranties for details.

© 2012 Allied Healthcare Products, Inc.® Printed in USA

*Customer Service:*

USA: (800) 444-3960  
 FAX: (314) 771-7537  
 Canada/Mexico: (800) 446-0552  
 FAX: (800) 246-6201  
 International: (314) 268-1683  
 FAX: (314) 771-5183



**Allied**  
HEALTHCARE PRODUCTS INC.

1720 Sublette Avenue  
 St. Louis, MO 63110 USA  
 (314) 771-2400  
 www.alliedhpi.com  
**ISO 9001: 2008**